REF VM 355-1

MEDICAL ADHESIVE B LIQUID

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Description

Medical Adhesive B Liquid is a pressure sensitive medical adhesive. It is particularly suitable for temporarily attaching various materials to the surface of the body. This medical adhesive offers the following advantages:

- It retains its adhesive properties even when it comes into contact with moisture
- Its adhesive properties remain unaffected by physiological temperature fluctuations
- Its adhesive properties do not decrease even over long periods of time

Suggestions for use

Medical Adhesive B Liquid is used as a non-sensitizing adhesive material to attach bandages, prostheses or other objects made of metal or plastic to the body.

Instructions for use

Spread the adhesive evenly on both surfaces to be glued (dry and free of grease). When dry to the touch, apply pressure to join the adhesive surfaces. Additional layers of adhesive can increase the strength of the bond.

Removing the glue

Medical Adhesive B Liquid can be removed with isopropanol alcohol or mineral spirits.

Storage and shelf life

Store container tightly closed at a maximum of 25 °C protected from light. If the container has been opened, use within 12 months. For shelf life see label.

Toxicology

Medical Adhesive B Liquid was tested for primary skin irritation. It was found to be non-toxic and non-irritating to the skin. Further information on the toxicology of this material can be found in our product information.

Attention

The adhesive contains ethyl acetate as a solvent, which can cause eye irritation. Therefore avoid eye contact. The solvent can cause irritation to the respiratory tract. Therefore avoid long and deep inhalation and ensure a sufficient supply of fresh air.

Important Note

The information and data presented here are based on information that we believe to be reliable. However, you should thoroughly test each intended use and decide independently whether our product meets your requirements. Do not smoke.

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AMT Medizintechnik GmbH
ULRICH AG